## **MRI Scanning Sanitization Protocol**

## **Before After During** ☐ Wear CNC provided PPE. ☐ Remove the keyboard and mouse covers. ☐ Wash hands upon entry into the ☐ Sanitize the area and anything that you touched with the ☐ Remain in the scanner suite. wipes that are provides. ☐ Use the sanitizing wipes and wipe all designated area until ☐ Wash hands and exit the MRI suite. the end of the scan. the surfaces, chair and chair handles. ☐ Meet the participant in the consent room if applicable or Research ☐ Place new covers on the eyetracker exit the center. computer keyboard and mouse. Staff ☐ Wash hands, wear gloves and face shield to remove participant from the scanner. ☐ Confirm the participant has the face mask before entering ☐ Leave the designated changing room the scanner room. ☐ Start the timer. Wait 20mins before entering the scanner unlocked. ☐ Wash hands upon entry into the room to sanitize the suite. scanner suite. ☐ Remove the covers on emergency squeeze ball and ☐ Place the covers on the scanner response box. ☐ Remove face shield and computer keyboard, mouse and ☐ Use the wipes provided to sanitize the inside of head coil, gloves. controller. headrest and padding. ☐ Remain in the ☐ Place covers for the emergency ☐ Use the UV light to sanitize the eyetracker mirror when designated area until **CNC Staff** squeeze ball and response box. necessary. the end of the scans. ☐ Place cover for the headrest and make ☐ Sanitize the inside of the scanner bore. the participant bed. ☐ Wipe all the door handles, metal wand, ear plug dispenser handle, surfaces and chairs/chairs handles. ☐ Wear mask, face shield and gloves. ☐ Remain in the designated spot until the ☐ Remove the scanner computer keyboard, mouse and participant is ready to be MRI screened. siemens controller covers. ☐ Wipe all the door handles and counter tops in the *changing* room and replenish the scrub packets. ☐ Wipe all the surfaces and door handles in the *consent room*.