

## MRI Scanning Sanitization Protocol

### Before

### During

### After



Research  
Staff 1  
Task handling

- Wear CNC provided mask.
- Wash hands when enter scanner suite.
- Place cover on the Siemens controller.

- Once everyone is out of the scanner room, ask the participant to pull down their mask.

- Sanitize the area and anything that you touched with the wipes that are provided.



Research  
Staff 2  
Participant handling

- Wear mask or CNC provided PPE.
- Wash hands upon entry into the scanner suite.

- Confirm the participant has their face mask on before entering the scanner room.
- Use the wipes provided to sanitize the inside of head coil, headrest (not the white one) and padding.
- Use the UV light to sanitize the eyetracker mirror and physio straps as needed (flip switch on, hit start, flip items, repeat).
- Sanitize the inside of the scanner bore with mop.
- Wipe all the door handles, metal wand, ear plug dispenser handle, surfaces and chairs/chair handles..
- Wipe all the door handles and counter tops in the *changing room* and replenish the scrub packets – if you use the last one of a size tell CNC staff.
- Export session to 'offline' from file browser window.